

Operation manual for Medicare Seatbath Clara Corner model

Articlenumber: 910150900

Medicare Seatbath Clara Wall model

Articlenumber: 910151000



Picture shows
Medicare Seatbath Clara
Corner model Art. No. 910150900



Security advices

Please read this operation manual carefully. Please use this unit only for the purpose it is designed for.

Please obey all remarks which are marked with "Attention".

Purpose of use

The **Medicare-Seatbath Clara** is designed for whole body seating bathes as well as for showering and giving relaxing bathes to persons which are able to do so.

Rules and regulations

All local rules and regulations have to be obeyed.

CE-Zeichen

The CE-sign refers to the regulation 93/42/EWG for Medical products.

Warranty

The manufacturer/importer of this unit is only responsible for security and proper working condition of the unit, if:

- A. The unit will be used according to its purpose.
- B. Frequent checks of function will be done thoroughly.

Transport damages

Please check the unit when delivered. Please react in case of transport damages according to your local regulations.

Serial number

The serial sign for the **Medicare-Seatbath Clara** is located as described in **Page 7, picture 3**.

The serial number can look like the following "example - 0020.16.00". Please always have this number available in case of questions.

Content

Security advices	Page	2
Purpose of use	Page	2
Rules and regulations	Page	2
Warranty	Page	2
Transport damages, Serial number	Page	2
Content	Page	3
Total view functions	Page	4
Total view and sizes Connections	Page	5
Technical data, models	Page	6
Installation, types	Page	6
First use / Operation	Page	6
Security lock / Serial sign	Page	7
Adjustment of the pre-adjusted Thermostatic fitting	Page	7
Warm airjet-whirl system	Page	8-9
Care, maintenance spare-parts	Page	10
EG-Conformity declaration (last page of this manual)		



Total view Operation valves

Picture 1

shows

Medicare-Seatbath Clara Corner model

Art. Nr. 910150900

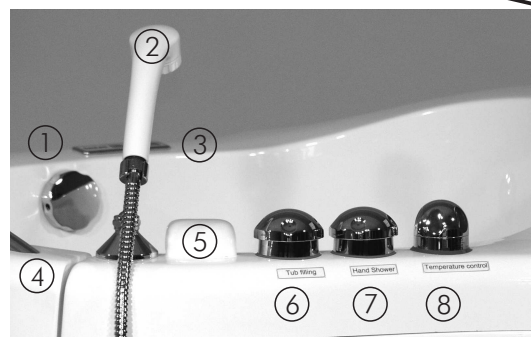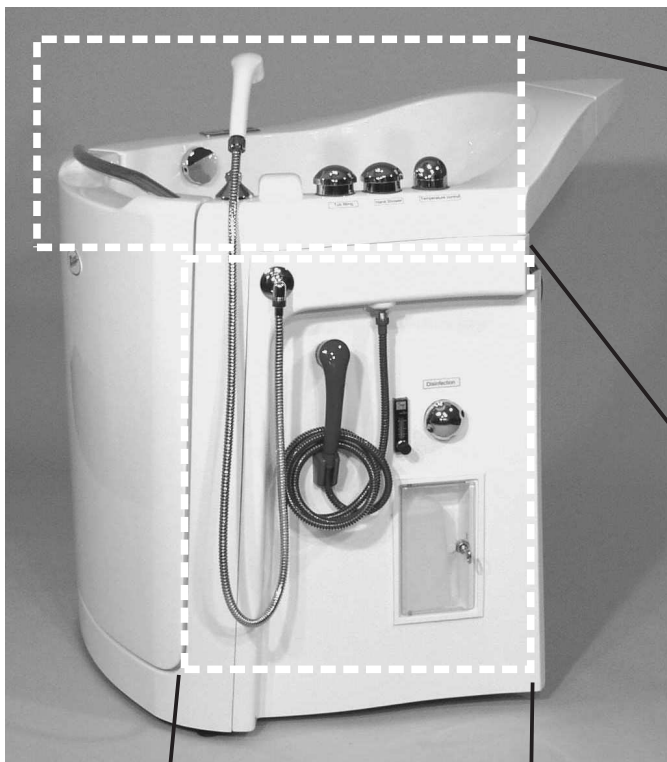

Picture 2

shows tub-filling



Picture 3

shows disinfection system

- ① Drainage opening/closing
- ② Hand shower
- ③ Control panel Air-jets
- ④ Hand grip door
- ⑤ Tub fill-tap PVC
- ⑥ Valve tub filling
- ⑦ Valve tub filling
- ⑧ Valve Thermostatic control
- ⑨ Connection shower hose
- ⑩ Disinfection shower
- ⑪ Flowmeter
- ⑫ Valve for disinfection
- ⑬ Disinfection box lockable


Total view and sizes

Picture 1

shows

Medicare-Seatbath Clara Corner model

Art. Nr. 910150900

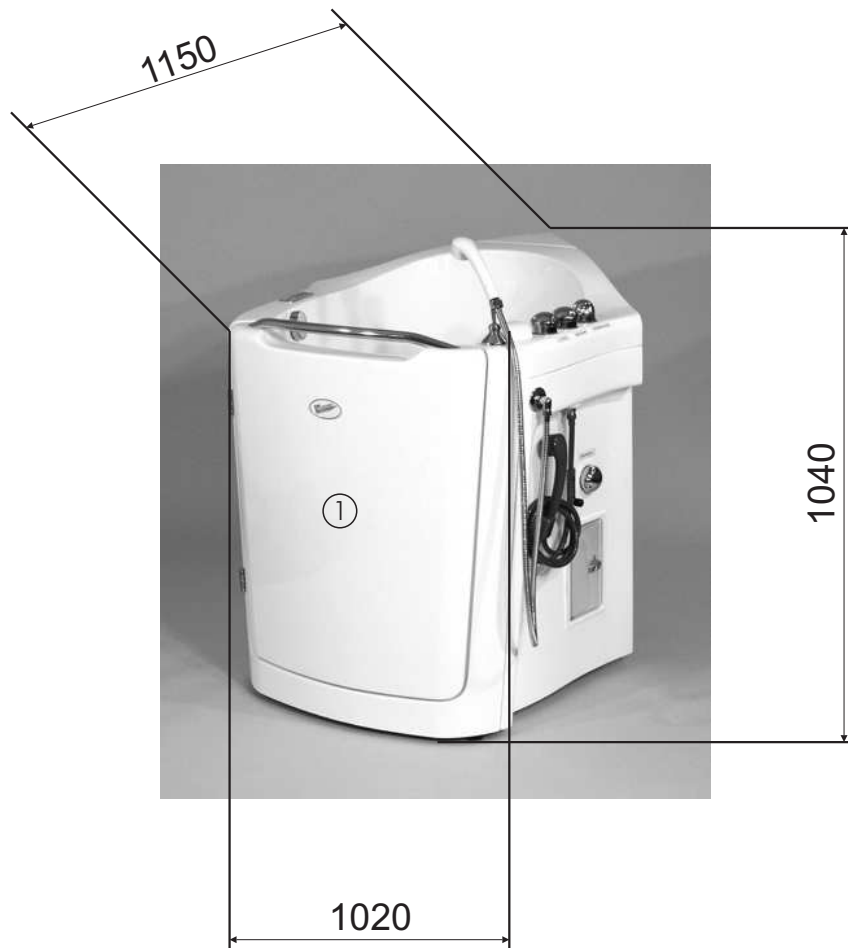

Weight:	265 lbs
Water content: approx.	44-55 gal.
Filling time at 3 Bar	
Pressure	approx. 3,5 Min.
Water drainage:	1,5 Min.
Cold water sleeve:	1/2"
Warm water sleeve:	1/2"

Pre-installed connection Sanitary

1. Warm water connection 1/2",
Inside thread tile-even
2. Cold water connection 1/2",
Inside thread tile-even
3. Ground connection, Drainage DN 50,
4. Wall connection, Drainage DN 50.

Pre-installed security

measures

Pre-closing valves Cold/Warm water
 Electro-installation according to the local safety
 rules for electric units in wet-rooms.

Technical Data

Length:	45 inches
Width:	39 inches
Height min:	41 inches
Weight w/o packaging:	ca. 65 lbs.

Installation

After installation of the water connection as well as after the installation of the electric connections, the **Medicare-Seatbath Clara** is ready to use.

Electric Connection (only at models with Warmair-Whirl system)

Please obey that all electric connection has to be done according to your local rules and regulations for electric connections.

Please allow only a instructed electrician to do the electric connections.

First use/operation Medicare-Seatbath Clara

After successful installation the **Medicare-Sitzbad Clara** is ready to be filled. To do so, please close the front door **Picture 1, page 5**. By lock of the security knob **Picture 1, page 7** the front door is secured. It can only be opened if the security knob is pulled and the lock will be opened.

ATTENTION

The front door of the Medicare-Seatbath Clara is protected by a security knob against accidentally opening. If the security knob is pulled and the locking mechanism will be unlocked, it is possible to open the door even with filled water.

ATTENTION

Please never open the door when tub is filled!!!

ATTENTION

The operation of the Medicare-Seatbath Clara is only allowed under survey of the staff.

Picture 1
shows security knob with locking mechanism




Picture 3
shows serial sign



Picture 2
shows Medicare-Seatbath Clara
Front door opened.



Serial sign

BEKA Hospitec GmbH & Co. KG Am Rübenmorgen 3 D-35582 Wetzlar Tel. 0641-92220-0		01	CE
Typ:			
Geräte Nr.:			
Nennsp.:	Frequenz:		
Nennaufn.:	Nennstr.:		

Models with warmair-whirl system

ATTENTION

To avoid any health problems by using this air-whirl system, the recommendation/permission of the doctors / nursing staff have to be obeyed.

The models with warm-air whirl systems have 12 built-in special air-jets with one central air-opening. All jets have an automatic self-closing function against water back flow in case of not operating or malfunction of the blower.

Function/operation

The control panel **picture 1, bottom** for the warm-air whirl system is located on the tub rim (seeing from front on the right side). The green LED-light indicates "stand-by - ready to use". For starting/stopping the whirl system please push the "on/off"- button **Picture 1, bottom**. A red LED-light indicates "system running". With the buttons + ② and - ③ (picture 1, bottom) the intensity of the blower can be adjusted individually.

Interval-function:

For activating this function, please push by working blower simultaneously the buttons ② and ③. By doing this, the blower output will vary periodically between minimum and maximum output. After activation, the period is 20 sec. By pushing the button ③ the period can be adjusted down to minimum 10 sec. By pushing the button ② up to maximum 40 sec. During the modulation the LED-lights (above button ② and ③) show the actual cycle.

Automatic Dry-blowing:

25 minutes after shut-off of the system, the system will turn on automatically for a period of 30 sec. During the waiting time of the up-coming dry-blowing of the air-jets, the LED-light (On/Off) is flashing.


Adjusting of the maximum blower out-put:

Please turn the system on. Adjust the blower output to the desired maximum power.

For saving this power, please push button ② and ③ simultaneously. The adjusted power will be saved as maximum blower out-put. For deleting this value, please push again the buttons ② and ③ simultaneously. In case of losing electric power,

Picture 1

shows control panel for warm-air whirl system

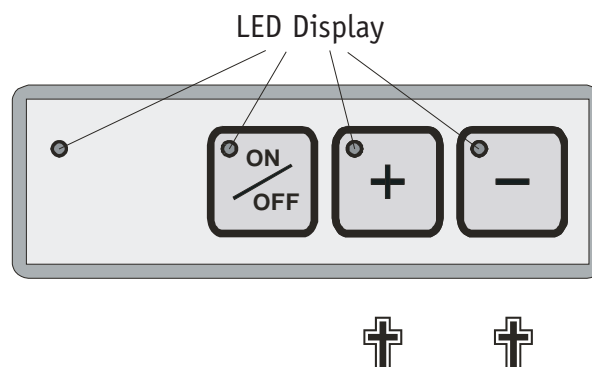

Terms of warranty

Dear customer

Thank you very much for buying this product. Please be informed that all products will be manufactured with highest care. However if you will find some disfunctions, please know:

Warranty on this tub is one year.

Warranty starts at delivery. If installation is to be completed by non-factory technician, prior approval is required.

No warranty by

- * Damages which happen by improper use and installation of this product.
- * Not obeying the local rules and regulations of the installation standards.
- * External damages (e.g. Transport damages).
- * Changings, interventions (e.g. At installation) which do not have the approval by the manufacturer.
- * Parts subject to wear and tear

Using and care advices

Disinfection system:

The adjustable disinfection system is located as shown at **page 4**.

How to use the tub disinfection system:

The disinfection fluid box **Nr.13** has to be filled. Please hold the red disinfection shower **Nr. 10** into the tub and open the disinfection valve **Nr. 12**. By turning the black knob at the flowmeter **Nr. 11**, the desired amount of disinfection fluid will be mixed automatically to the water coming out of the red hand shower. (This adjustment must be done usually only at first use - depending on the different disinfection fluids which may be used.) The amount of the mixture as well as the disinfection period is following the rules of the disinfection producer. Please always follow their instructions. After finishing disinfection process, always rinse with clear water.

ATTENTION

**The disinfection of the tub must be done without patients in the tub.
The instructions of the disinfection producer must be followed.**

Care of the tub body

The tub body of the **Medicare-Seatbath Clara** can be cleaned with any non-scratching fluid detergent. We advice to polish the glasfibre every two months. This allows the tub to stay in good shape many years.

ATTENTION

Never use a scratching detergent for cleaning the tub

Fittings

All metallic as well as colored surfaces are non-resistant against acid and scratching detergents. Therefore please do not use scratching detergents for cleaning the fittings.

Bathing ingredients

By using this tub as a normal bath tub there are no objections for adding any ingredients.

ATTENTION

**By using the Air-Whirl system , no foaming ingredients are allowed.
Those ingredients could close the jet openings and damage the blower.**

Using the Whirl-system

The advised bathing temperature is between 96,8 - 100°F, period of 10 - 20 minutes. However always follow the instructions of the nursing staff / therapist or medical doctor.

Maintenance

1. Frequent checks of all connections and installation have to be done.
2. Every 4 weeks the locking mechanism of the front door has to be checked.
3. The "coal brushes" inside of the blower have to be replaced every 1000 working hours.
Please consult your local dealer.

In case of questions

Please contact your local dealer for any questions you may have. Please always have the serial number available for immediate reaction.